


## Redefining the knotless suture anchor market again!





- **Low-profile knotless repair** provided by proven reproducible technique
- **Superior tissue shift** with Active/Controlled tension of the labrum
- **NEW - Blue color** improves operative contrast and visualization
- **NEW - Retention feature** provides operative loop security during transition from loop capture to anchor insertion

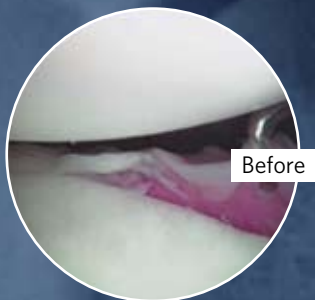

# BIOKNOTLESS™ PLUS

## ANCHOR

# BIOKNOTLESS™ PLUS ANCHOR

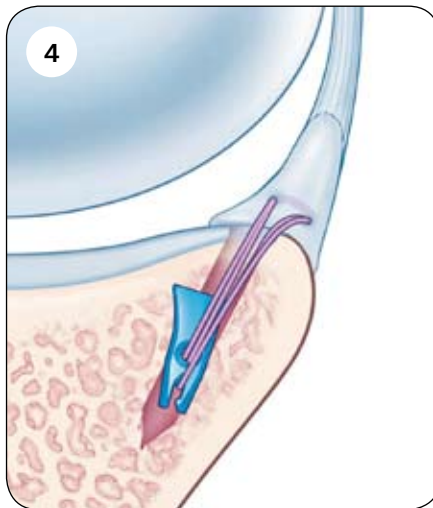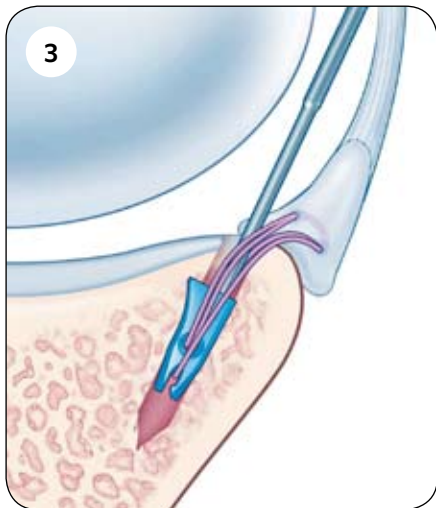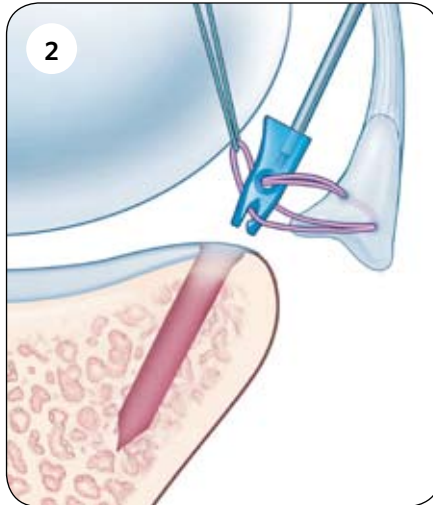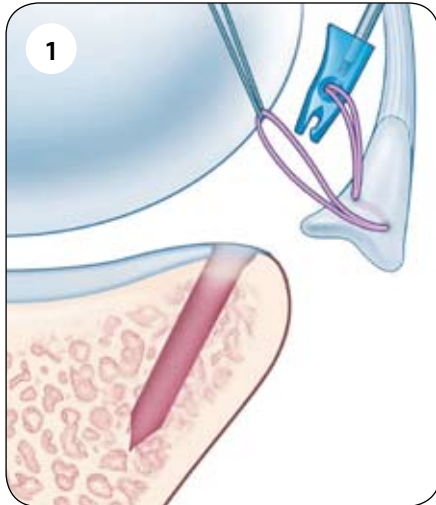

## SURGICAL TECHNIQUE

1. Pull the anchor loop through the tissue and capture one strand in the mouth of the BIOKNOTLESS Anchor.
2. Orient anchor straight into the drill hole.
3. Insert the BIOKNOTLESS Anchor into the drill hole.
4. Once the tissue is at the desired location on the bone, unscrew the inserter and pull the utility loop free from the anchor.

## ORDERING INFORMATION

Catalog No.	Description	Order Qty.
212724	BIOKNOTLESS Plus with ETHIBOND®	
212725	BIOKNOTLESS Plus with PANACRYL®	
212726	BIOKNOTLESS Plus with OROTHOCORD™	



For more information, call your DePuy Mitek representative at 1-800-382-4682 or visit us at [www.depuymitek.com](http://www.depuymitek.com). DePuy Mitek, Inc., 325 Paramount Drive, Raynham, MA 02767.

© DePuy Mitek, Inc., 2007. All rights reserved. Printed in the USA. P/N 900950 6/07  
Dyneema® Purity is a trademark of Royal DSM N.V.

